



Sure Ventt2®

Sure Vent 2™ Precautions and Warnings

- The SV2™ should be used only by individuals who have adequate training in the operation of gas-powered resuscitators.
- Do not use grease or oil on the SV2™ for any reason
- Never use the SV2™ in oxygen deficient atmospheres or near open flames
- Do not smoke while using the SV2™ or any other oxygen equipment
- Governing law restricts the use of this device to sale by or on the order of a physician or properly licensed practitioner and for **single patient use only.**
- Do not dismantle or remove any components other than allowed within the operation guidelines (please see operation guidelines).
- Patients connected to the SV2™ are to be monitored continuously by persons having appropriate training.
- Do not leave patients unattended.
- Do not use the SV2™ as an unattended automatic ventilator.